NOTE DE SERVICE

MEMORANDUM

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As part of its continuous improvement efforts and commitment to quality, Optilab Montréal-MUHC would like to remind all of its external clients of its specimen conservation and transport requirements.

To ensure that specimen stability and integrity is maintained, allowing us to produce results of the highest quality, the following requirements must be respected:

Specimen Type/Test	Acceptable Delay of Transport
EDTA (Lavender) – <i>CBC</i>	Must be received in lab within 3 hrs of collection
Sodium Citrate (Blue) – <i>Coagulation Profile</i>	Must be received in lab within 3 hrs of collection
10 ml Conical Tube – <i>Urinalysis</i>	Must be received in lab within 3 hrs of collection
SST (Yellow with gel) OR Lithium Heparin (Mint Green with gel) – <i>Chemistry, Endocrinology, Serology, etc</i>	Stabilized* specimens must be received in lab within shortest delay possible Unstabilized specimens (non-centrifuged SST tubes) must be received in lab within 1 hr of collection.

*It is highly recommended that all SST tubes be centrifuged and stabilized prior to transport. Centrifuge at 3,500 RPM for 10 minutes following the centrifuge's User Manual. Unstabilized specimens must be received in the lab **within 1 hr of collection.**

As of **Monday, September 19, 2022**, specimens that are received in the laboratory outside of the acceptable delay in transport will be rejected as we cannot ensure the integrity of the specimen and therefore the accuracy of the results.

In order for us to validate specimen stability and integrity it is mandatory for the **collection date and time** to be recorded on the test requisition. If this information is not recorded, it will result in the rejection of the specimen.

We thank you in advance for your cooperation in helping us ensure that we are providing quality services to all of our clients and patients.