NOTE DE SERVICE

MEMORANDUM

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This is a reminder that the Central Laboratory – Glen Site will only be accepting urine specimens for **urinalysis testing** if they are received in the appropriate conical tube. This applies to urinalysis test requests only. Please note that this does not affect a urine specimen for a **urine culture** test which must continue to be sent in a separate **sterile urine container** to the **Microbiology laboratory**.

Urine specimens that are not sent in the appropriate conical tubes will be rejected.

ACCEPTABLE conical tube for urinalysis

NOT ACCEPTABLE for urinalysis – Central Laboratory

Appropriate container for urine culture specimens – Microbiology Laboratory

Product Information

The following conical tubes are the **ONLY ACCEPTABLE** urine containers as these are the only ones that have been validated on the urinalysis instrument. No other alternative will be accepted.

Conical Tube Order			
Internal Clients - MUHC		External Clients (one of the following options)	
Logibec Order	GRM #1002313	Sarstedt	Ultident Scientifique
		Code : 62.9924.293	Code : 11-6293
		Conical tube 10 ml with screw cap	

Urine Collection Instructions for Urinalysis Tests

- Have the patient collect a urine specimen in a clean plastic/styrofoam cup and ask them to transfer the urine into the conical tube.
- If the patient also has a urine culture test to do at the same time, follow the instructions for a sterile urine collection. The urine specimen can be directly collected in the sterile urine container and some of it be transferred to the urine conical tube.
- Ensure the cap is properly closed to avoid specimen leakage.
- Affix the patient identification/barcode label to the conical tube and send the specimen to the Central Laboratory as soon as possible (< 3 hours). Urine culture specimens must be collected in a sterile urine cup and sent directly to the Microbiology laboratory.
- The urine specimen for a urinalysis must be delivered to the Central Laboratory within **3 hours** of collection.

If you have any questions, please contact Laboratory Client Services at 514-934-1934 ext 35687.